Daumenschiene

Thumb Splint

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Thumb Splint

- Soft brace with practical lacing
- Soft inner padding

Many thanks for placing your trust in this medical device. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

INTENDED PURPOSE

This medical device is a brace for immobilisation of the metacarpophalangeal joint and saddle joint of the thumb.

INDICATIONS

Post-operative, post-traumatic, rhizarthrosis, degenerative, inflammatory, e.g. arthritis, ulnar/radial lesion of thumb ligaments.

CONTRAINDICATIONS

Peripheral arterial occlusive disease (PAOD), lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders with the region of the body treated, skin diseases in the part of the body treated.

APPLICATION RISKS/IMPORTANT NOTES

This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.

- remove the medical device prior to radiological examinations
- remove it during longer periods of rest, e.g. when sleeping
- in case of numbness, loosen the medical device and remove it if necessary
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds

- do not use in case of intolerance of one of the materials used
- no re-use – this medical aid is intended for treating one patient
- whilst wearing the brace, please neither use any creams nor ointments on or around it as they can destroy the material

FITTING INSTRUCTIONS

- 1 Before putting it on, set the thumb wrap and the wrist strap to the largest possible circumferences and pull the thumb splint over the thumb and the metacarpus.
- 2 Firstly, set the thumb wrap and next the wrist cord strap which can be set to the individual width using the Velcro surfaces.
- 3 Close the support completely.

REMOVAL

To remove the support, please open the Velcro fasteners and slip the thumb brace off.

MATERIAL COMPOSITION

Polyamide (PA), elastane (EL), polyurethane (PU), polyester (PES), viscose (CV)
The sewn in textile label provides the precise material composition.

CLEANING INFORMATION

 Delicate wash 30 °C  Do not bleach
 Do not dry in a tumble dryer
 Do not iron  Do not clean chemically
 Do not use fabric conditioner. Dry in the fresh air.

GUARANTEE

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

USEFUL LIFE/LIFETIME OF THE PRODUCT

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

DUTY OF NOTIFICATION

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA).

You can find the manufacturer's contact data in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: www.bort.com/md-eu-contact.

DISPOSAL

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

DECLARATION OF CONFORMITY

The manufacturer confirms that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL.

Status: 03.2022

Medical device |  Single patient – multiple use

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VPS Medical Ltd.
BORT GmbH | Am Schweizerbach 1 | D-71384 Weinstadt